VanishPoint® Allergy Syringe Tray

Automated Retraction Technology

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PATENTS—WWW.RETRACTABLE.COM / PATENTS




© 2000, 2020. Retractable Technologies, Inc. All rights reserved.

VanishPoint® Allergy Syringe Tray

Automated Retraction Technology

VanishPoint® Behälter für Allergie-Injektionsspritzen
Plateau à seringues pour allergies VanishPoint®
Vassoio per siringhe per prove allergiche VanishPoint®
Bandeja de jeringas para alergia VanishPoint®
VanishPoint® Set injectiespuiten tegen allergie

Bandeja de seringas de alergia VanishPoint®
VanishPoint® アレルギー注射器トレイ
VanishPoint® Alerji Şırıngası Tepsisi
Taca strzykawk do alergii VanishPoint®

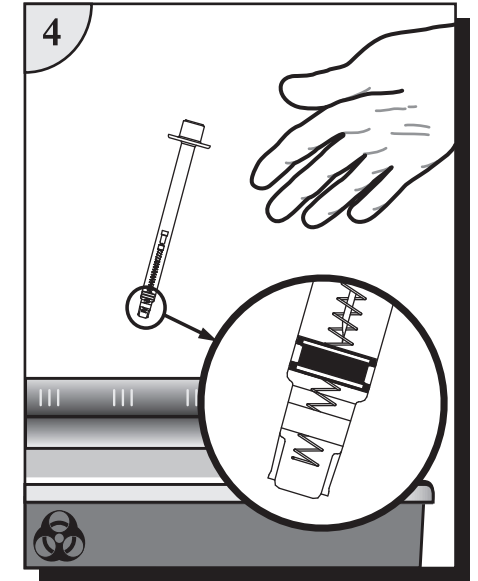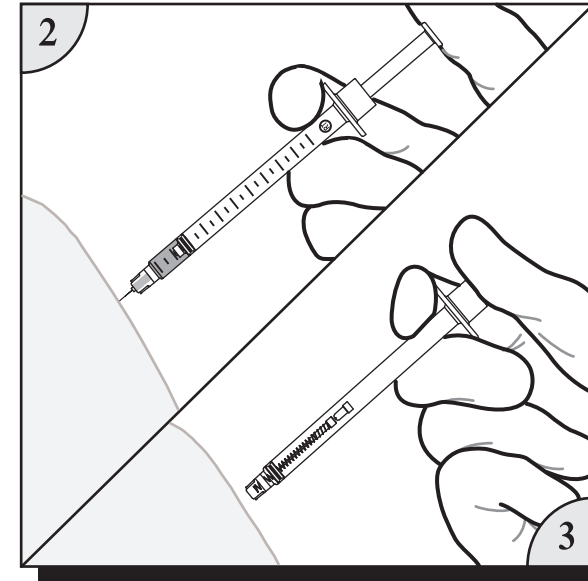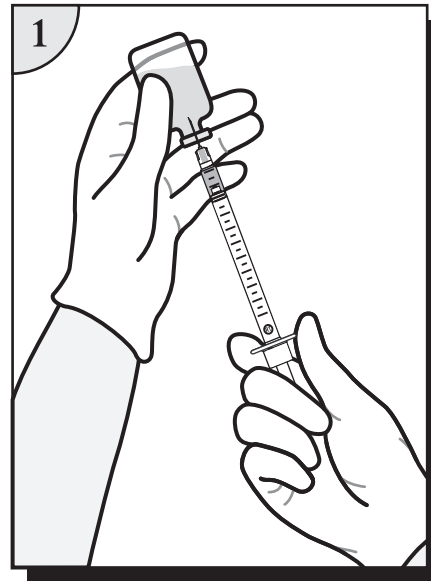Precautions:

- Single use only. Reuse of this device may result in exposure to bloodborne pathogens, including Hepatitis B virus (HBV), Hepatitis C virus (HCV), and human immunodeficiency virus (HIV).
- Contents are sterile, non-toxic, and non-pyrogenic.
- Contents are sterile until package is opened. Do not use if product or package is damaged.
- Not made with natural rubber latex.
- Use only with attached needle. Needle cannot be changed.
- Automated needle retraction occurs only when barrel is emptied and plunger is fully depressed.
- For applications where full dose is not administered, expel remaining contents according to institutional policy and activate needle retraction.
- U.S. Federal Law restricts this device to sale by or on the order of a physician.

Instructions for Use:

1. Prepare and give injection using aseptic technique according to institutional policy.
2. For injection into patients, continue depressing plunger to activate automatic needle retraction **while needle is still in patient. Full dose is administered only when needle retraction is activated.**
3. Needle will automatically retract into syringe, preventing exposure to contaminated needle and rendering syringe non-reusable. In the event that needle retraction mechanism does not activate, discard syringe in an appropriate sharps container per protocol of institution. Do not recap contaminated needles.
4. Dispose of VanishPoint® syringe in an appropriate sharps container per protocol of institution.

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