

# VanishPoint<sup>®</sup> Syringe

Automated Retraction Technology

VanishPoint<sup>®</sup> Injektionsspritze

Automatisierte Retraktionstechnologie

Seringue VanishPoint<sup>®</sup>

Technologie de Rétraction Automatisée

Siringa VanishPoint<sup>®</sup>

Tecnologia di Retrazione Automatizzata

Jeringa VanishPoint<sup>®</sup>

Tecnologia de Retracción Automatizada

VanishPoint<sup>®</sup> Injectiespuit

Geautomatiseerde Retractietechnologie

Seringa VanishPoint<sup>®</sup>

Tecnologia de Retração Automatizada

VanishPoint<sup>®</sup> シリンジ

自動引き込み技術

VanishPoint<sup>®</sup> Şırınga

Otomatik Geri Çekme Teknolojisi

Strzykawka VanishPoint<sup>®</sup>

Technologia Automatycznego Wycofywania

DD-CL032, Rev. 1

# VanishPoint<sup>®</sup> Syringe

Automated Retraction Technology

VanishPoint<sup>®</sup> Injektionsspritze

Seringue VanishPoint<sup>®</sup>

Siringa VanishPoint<sup>®</sup>

Jeringa VanishPoint<sup>®</sup>

VanishPoint<sup>®</sup> Injectiespuit

Seringa VanishPoint<sup>®</sup>

VanishPoint<sup>®</sup> シリンジ

VanishPoint<sup>®</sup> Şırınga

Strzykawka VanishPoint<sup>®</sup>






Corporate Headquarters  
Retractable Technologies, Inc.  
511 Lobo Lane, P.O. Box 9  
Little Elm, Texas 75068-0009  
www.retractable.com

Sales and Marketing  
Retractable Technologies, Inc.  
511 Lobo Lane, P.O. Box 9  
Little Elm, Texas 75068-0009  
Fax: (972) 294-4400  
Toll Free 1-888-703-1010

EU Auth. Rep.  
(MDD93/42/EEC)  
MDSS GmbH  
Schiffgraben 41  
D-30175 Hannover  
Germany

Manufacturing Location  
Jumin Bio-Technologies Co., Ltd.  
1888 Huhang Road,  
Jianghai Economic Zone  
Fengxian, Shanghai  
P. R. China

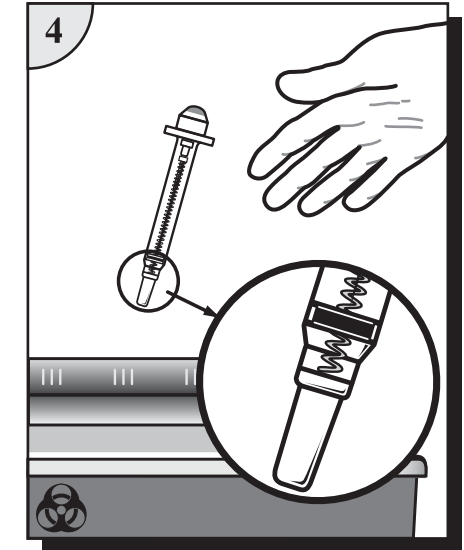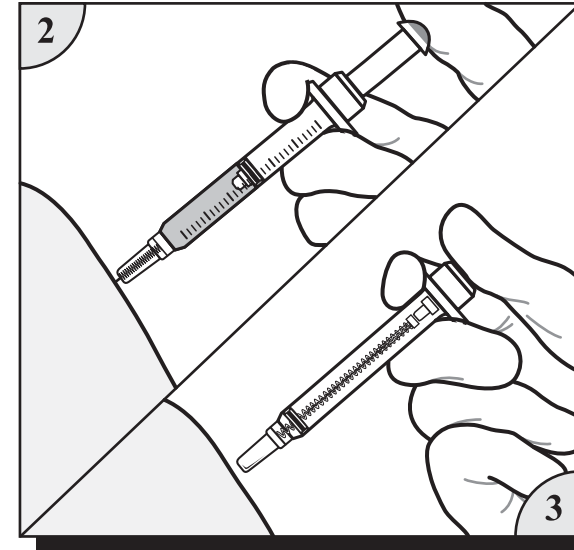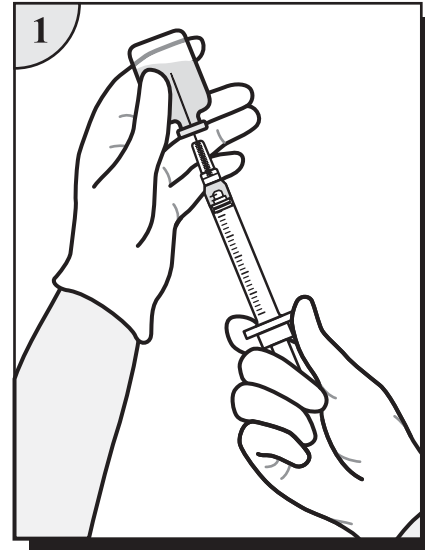

### Precautions:

- Single use only. Reuse of this device may result in exposure to bloodborne pathogens, including Hepatitis B virus (HBV), Hepatitis C virus (HCV), and human immunodeficiency virus (HIV).
- Contents are sterile, non-toxic, and non-pyrogenic. Do not use if product or package is damaged.
- Not made with natural rubber latex.
- Use only with attached needle. Needle cannot be changed.
- Automated needle retraction occurs only when barrel is emptied and plunger is fully depressed.
- For applications where full dose is not administered, expel remaining contents according to institutional policy and activate needle retraction.
- U.S. Federal Law restricts this device to sale by or on the order of a physician.

### Instructions for Use:

1. Prepare and give injection using aseptic technique according to institutional policy.
2. For injection into patients, continue depressing plunger to activate automatic needle retraction **while needle is still in patient**.  
For injection into IV ports, continue depressing plunger to activate automatic needle retraction and **immediately remove needle from port. Full dose is administered only when needle retraction is activated.**
3. Needle will automatically retract into syringe, preventing exposure to contaminated needle and rendering syringe non-reusable. In the event that needle retraction mechanism does not activate, discard syringe in an appropriate sharps container per protocol of institution. Do not recap contaminated needles.
4. Dispose of VanishPoint<sup>®</sup> syringe in an appropriate sharps container per protocol of institution.

PATENTS--WWW.RETRACTABLE.COM/PATENTS

© 2000, 2022. Retractable Technologies, Inc. All rights reserved.

