

VanishPoint® II Insulin Syringe

Automated Retraction Technology

VanishPoint® II Insulinspritze

Automatisierte Retraktionstechnologie

Seringue à insuline VanishPoint® II

Technologie de Rétraction Automatisée

Siringa da insulina VanishPoint® II

Tecnologia di Retrazione Automatizzata

Jeringa para insulina VanishPoint® II

Tecnología de Retracción Automatizada

VanishPoint® II Insulinespuit

Geautomatiseerde Retractietechnologie

Seringa de insulina VanishPoint® II

Tecnologia de Retração Automatizada

VanishPoint® II インスリンシリンジ

自動引き込み技術

VanishPoint® II İnsülin şiringası

Otomatik Geri Çekme Teknolojisi

Strzykawka insulinowa VanishPoint® II

Tecnologia Automatycznego Wycyfowania

VanishPoint® II Insulinspruta

Teknologi för automatisk tillbakadragning av nål

U-100

REAL SAFETY™



RETRACTABLE®
TECHNOLOGIES, INC.
www.retractable.com

U-100



Corporate Headquarters
Retractable Technologies, Inc.
511 Lobo Lane, P.O. Box 9
Little Elm, Texas 75068-0009
USA
www.retractable.com

Sales and Marketing
Retractable Technologies, Inc.
511 Lobo Lane, P.O. Box 9
Little Elm, Texas 75068-0009
Fax: (972) 294-4400
Toll Free 1-888-703-1010

EU Auth. Rep.
(MDD93/42/EEC)
MDSS GmbH
Schiffgraben 41
D-30175 Hannover
Germany




Manufacturing Location
Jumin Bio-Technologies Co., Ltd.
1888 Huhang Road
Jianghai Economic Zone
Fengxian, Shanghai
P.R. China

PATENTS—WWW.RETRACTABLE.COM/PATENTS

© 2022, Retractable Technologies, Inc. All rights reserved.

VanishPoint® II Insulin Syringe

Automated Retraction Technology

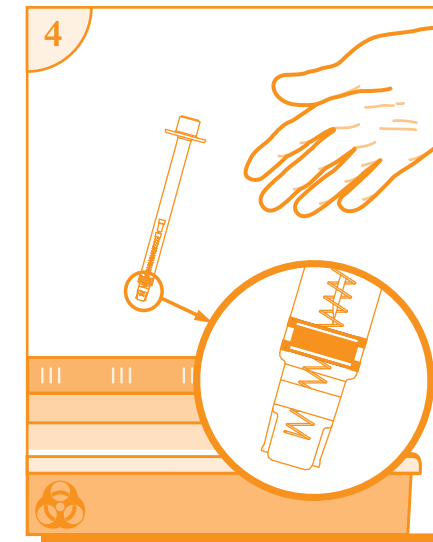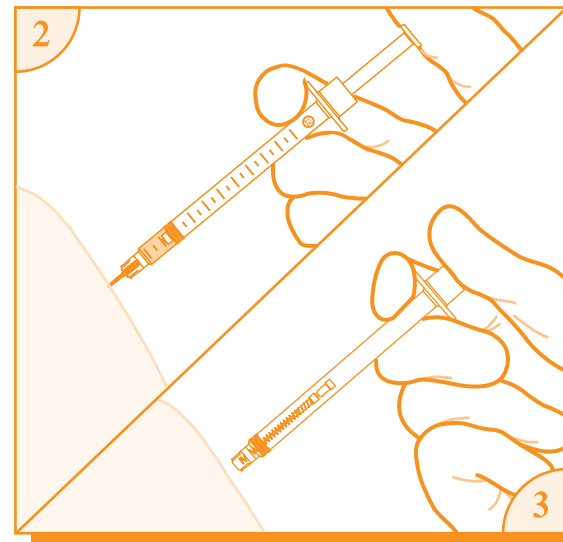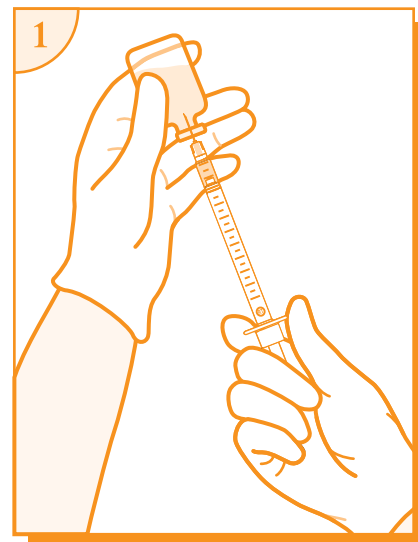

Precautions:

- Use U-100 Insulin only.
- Single use only. Reuse of this device may result in exposure to bloodborne pathogens, including Hepatitis B virus (HBV), Hepatitis C virus (HCV), and human immunodeficiency virus (HIV).
- Contents are sterile, non-toxic, and non-pyrogenic. Do not use if product or package is damaged.
- Not made with natural rubber latex.
- Use only with attached needle. Needle cannot be changed.
- Automated needle retraction occurs only when barrel is emptied and plunger is fully depressed.
- U.S. Federal Law restricts this device to sale by or on the order of a physician.

Instructions for Use:

1. Prepare and give injection using aseptic technique according to institutional policy.
2. Continue depressing plunger to activate automatic needle retraction **while needle is still in patient. Full dose is administered when the plunger handle is fully depressed**, which activates the needle retraction mechanism.
3. Needle will automatically retract into syringe, preventing exposure to contaminated needle and rendering syringe non-reusable. In the event that needle retraction mechanism does not activate, discard syringe in an appropriate sharps container per protocol of institution. Do not recap contaminated needles.
4. Dispose of VanishPoint® II syringe in an appropriate sharps container per protocol of institution.

