


Needlestick prevention that meets your needs

- Pre-removal activation virtually eliminates exposure
- Single-handed, passive activation
- Hands remain behind the sharp
- Integrated safety mechanism
- Prevents needle removal
- Remains safe through disposal
- Easy to use
- Safe and effective for patients

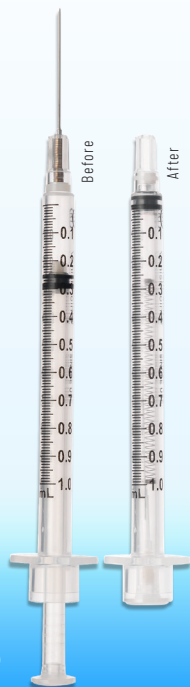

SAFE FOR YOU AND YOUR PATIENTS

A study comparing the effectiveness of passive SED (safety-engineered devices) with that of active SED, concluded that passive SED (such as the VanishPoint® syringe) significantly reduced the needlestick injury rate.*

ATTACHED NEEDLE DESIGN:

- Reduces deadspace
- Maintains a closed system
- Prevents leakage and contamination
- Prevents unnecessary needle changes

*A. J. Goris, et al., "Reducing Needlestick Injuries from Active Safety Devices A Passive Safety-Engineered Device Trial," AOHJ Journal, vol. 34, no. 2(Spring 2014) pp. 14-18.

CATALOG

No.	Description	
VanishPoint® U-100 insulin syringes 100 per box/800 per case		
15391	0.5mL VanishPoint insulin syringe (half-unit)	30G x 3/16"
15371	0.5mL VanishPoint insulin syringe (half-unit)	30G x 5/16"
15291	0.5mL VanishPoint insulin syringe	30G x 3/16"
15271	0.5mL VanishPoint insulin syringe	30G x 5/16"
15221	0.5mL VanishPoint insulin syringe	30G x 1/2"
10291	1mL VanishPoint insulin syringe	30G x 3/16"
10271	1mL VanishPoint insulin syringe	30G x 5/16"
10281	1mL VanishPoint insulin syringe	29G x 5/16"
10211	1mL VanishPoint insulin syringe	29G x 1/2"
1mL VanishPoint® syringes 100 per box/800 per case		
10131	1mL VanishPoint syringe	27G x 1/2"
10151	1mL VanishPoint syringe	25G x 5/8"
10161	1mL VanishPoint syringe	25G x 1"
VanishPoint® Allergy Syringe Tray 25 per tray/1000 per case		
10134	1mL VanishPoint allergy syringes	27G x 1/2"
3mL VanishPoint® syringes 100 per box/600 per case		
13021	3mL VanishPoint syringe	27G x 1 1/2"
10301	3mL VanishPoint syringe	25G x 5/8"
10391	3mL VanishPoint syringe	25G x 1"
13011	3mL VanishPoint syringe	25G x 1 1/2"
10311	3mL VanishPoint syringe	23G x 1"
10321	3mL VanishPoint syringe	23G x 1 1/2"
10331	3mL VanishPoint syringe	22G x 1"
10341	3mL VanishPoint syringe	22G x 1 1/2"
10351	3mL VanishPoint syringe	21G x 1"
10361	3mL VanishPoint syringe	21G x 1 1/2"
10371	3mL VanishPoint syringe	20G x 1"
10381	3mL VanishPoint syringe	20G x 1 1/2"
5mL VanishPoint® syringes 100 per box/600 per case		
10mL VanishPoint® syringes 100 per box/600 per case		

5mL and 10mL VanishPoint® syringes are available with a variety of needle lengths and gauges. For a complete list, please contact RTI or visit our website at www.retractable.com



RETRACTABLE
TECHNOLOGIES, INC.®

rtisales@retractable.com


511 Lobo Lane, Little Elm, Texas 75068-5295 USA
Tel.: (972) 294-1770, Fax: (972) 294-4400
Toll Free: 1-888-703-1010



www.retractable.com



The NEW standard for Real Safety™

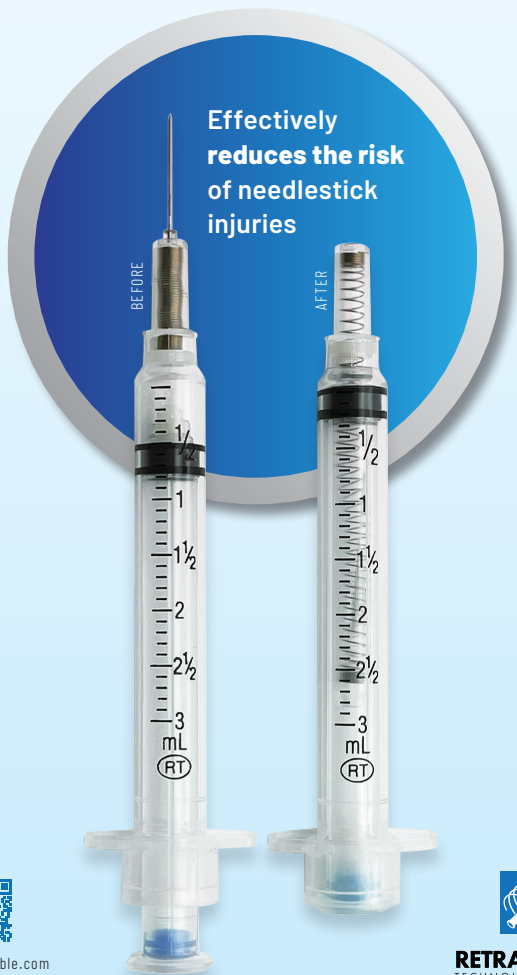

www.retractable.com



RETRACTABLE
TECHNOLOGIES, INC.®



VanishPoint[®]

ADVANTAGE

1

Use standard procedure for drawing up medication.



2

Prepare and give injection using aseptic technique according to institutional policy.



3

While the needle is still in the patient, fully depress the plunger to activate retraction.



4

Dispose of syringe in an appropriate sharps container.




Effective safety with automated retraction technology



Triple beveled, lubricated needle provides patient comfort

Clear, unobstructed Graduations allow for accurate dosing

EASY, ONE-HANDED ACTIVATION

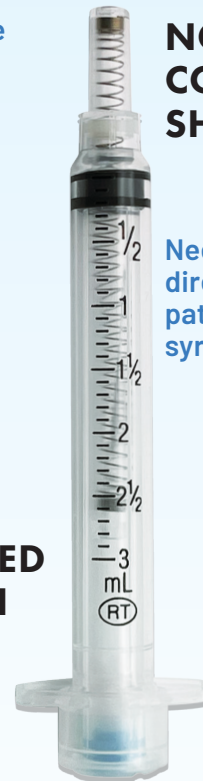

NO CONTAMINATED SHARP EXPOSED

Needle retracts directly from patient into syringe

SAVES MONEY, TIME & LIVES

NEW DESIGN
IMPROVED FOR YOU

Same Retraction and Safety!



- **New Plunger Handle Design** for increased user comfort and control
- **Enhanced Bubble Removal** for quick and easy medication preparation

DISPOSAL SAVINGS



VanishPoint[®] SYRINGES

Competitor

Activated VanishPoint[®] syringes require less disposal space than other safety syringes and prevent disposal-related injuries.